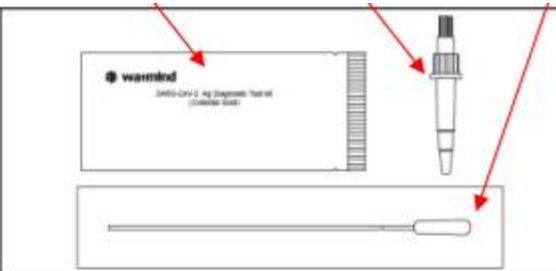
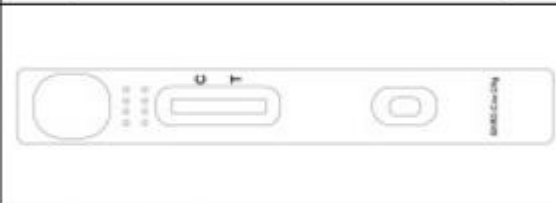
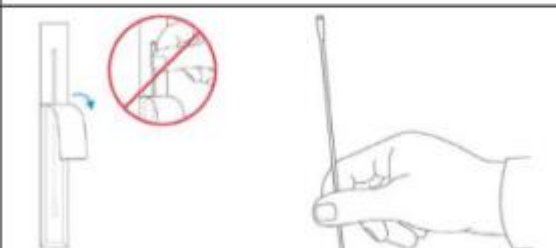
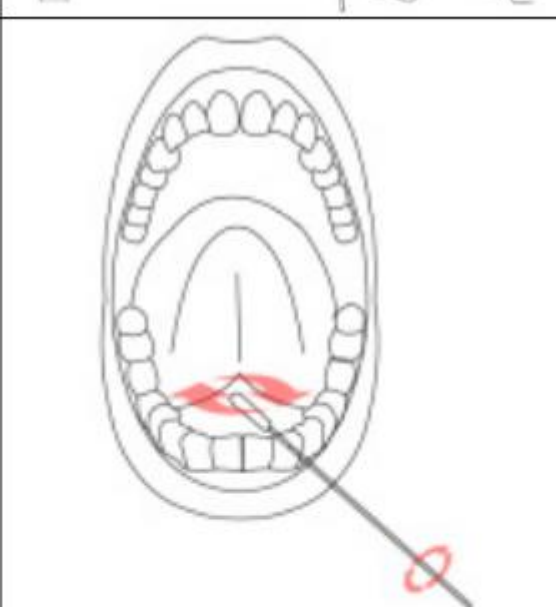

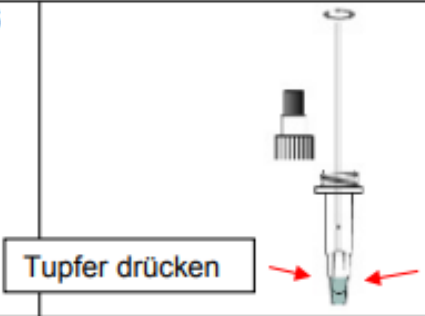



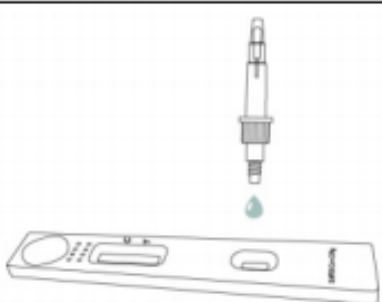


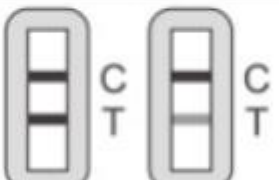
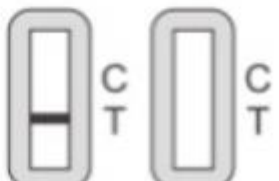


	Testkassette	Röhrchen	Tupfer=Lolli
1			
			<p>1) JU NEVOITET: 1x KASET PROVE NE QESE PETE´</p> <p>1x TUBA ME LE´NG</p> <p>1x SHTUPE´</p>
2			
			<p>2) HAPNI QESEN E FLETES DHE VENDOSNI KASETEN NENJE TRUEZE´ TE PASTER DHE TE´</p> <p>THAT</p> <p>SHENIMET: MOS PREKNI ASNJE PJES NE KASET</p>
3			
			<p>3) HAPNE JU SHTUPE</p> <p>SHENIMET: SHTUPE MOS E PREKNI NE FUND</p>
4			
			<p>4) 30 MINUTA PARA PERDORIMIT, NUK DUHINI, ME NGREN, ME PI, MI PASTRU</p> <p>DHEMBET, QAMSAKEZ: BONBONZ AS GJE TE HANI</p> <p>SHTUPE NE GOJ:</p> <p>GJUHEN E NGRENI MRAPA NEL E SHYTPNI, PESHTYMA MBLIDHET NEN GJUHE SHTUPEN</p> <p>10 SEKONDA NE PSHTYM MBANE, TANJ SJELLNE DERI KUR LAGET KREJT TESTIN BEJNI</p> <p>MENJEI</p>
5			
			<p>5) TUBI KA DY KAPAKE</p> <p>KAPAKU I POSHEM ESHT ITRASHE DE VJOLLZE</p> <p>KAPAKU I LART ESHT I HOLL DHE NGJYR E BAROH</p> <p>ME SE PARI KAPAKUN VJOLLE HEKNE (HAPNE)</p>

Übersetzung Handhabung „Lolli-Test“ in ALBANISCH Sprache

<p>6</p>		<p>VENDOSNE PSHTYMEN NE TUB</p> <p>TUBEN NE UJ FUTNE</p> <p>SHTUPEN NE UJ SHTYNE DHE SILLNE</p> <p>SHTUPEN NE UJ LENE</p> <p>SHTRYLLNI TUBIN NE GISHTABASHK</p> <p>SHTRYLLNTE PIKON NEPERTE´</p>
<p>7</p>		<p>7) PERKULNI SHTYPNI DHE VENDOSNI GOTEN NE TUB</p>
<p>8</p>		<p>8) PE´R TE´ BERE´ TUBIN, RRETILLONI TUBIN 5 HERE´</p>
<p>9</p>		<p>9) HAPNEN KAPAKUM E HOLLTE BARDHIN TE´ TUBIT</p>
<p>10</p>		<p>9) HAPNEN KAPAKUM E HOLLTE BARDHIN TE´ TUBIT</p>

Übersetzung Handhabung „Lolli-Test“ in ALBANISCH Sprache

<p>11</p>		<p>15 MINUTA PRITNI</p> <p>SHENIMET: KASETE MOS E LEVIZNI</p> <p>KQYRNE OREN</p> <p>MOS E LENI MA GJAT SI 15 MINUTA</p>
<p>12</p>		<p>REZULTATI NEGATIV - C</p>
<p>13</p>		<p>REZULTATI POZITIV - C</p>
<p>14</p>		<p>REZULTANTI I PAVLEFSHME - T</p>